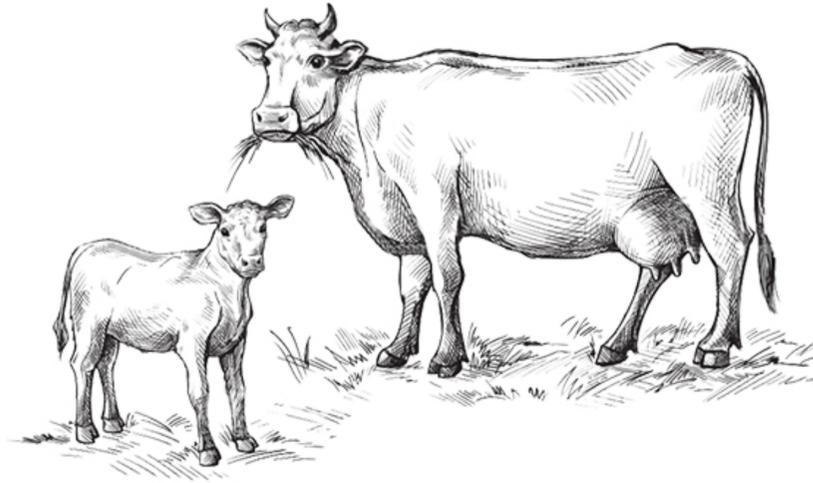
Pregnancy detection in

**Whole blood/serum/plasma
28-day serum/plasma/35-day whole blood**

Bignano


Ringbio

bovine pregnancy rapid test kit


Operations

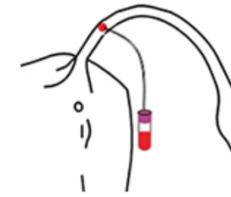
- 1) Read the instructions before experiment. Serum samples should be freshly collected or should be frozen at -20°C or below. Bring the test kit and the test samples to room temperature ($20-30^{\circ}\text{C}$).
- 2) Take test card needed from the kit package, and make proper marks. Please use the test strips within 1h. Return the rest into cold storage.
- 3) Take 60ul serum/plasma sample into the sample well on the test card, then add 60ul (2 drops) of sample buffer with the buffer vial in the kit. Make sure the buffer vial is empty after adding.
- 4) Wait for 15min and then determine the result according to Part 4. Results observed after 30min is invalid.

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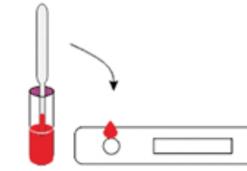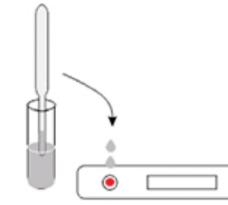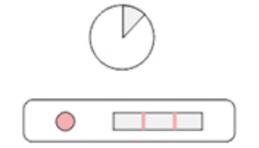
Result determination

There are 2 lines on the test card, Control Line, Test Line. Test results will depend on the color of these lines.
Positive (+): Line C and line T are visible and red.
Negative (-): Line C is red, and Line T is not visible.
Invalid (x): Line C has no color. In this case, please read the instructions and repeat assay with new test card.
Incomplete control line or incomplete test line is still VALID.
Note: positive result needs to be confirmed by testing of sample from the cow on the next day.

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