DCM092-6
Ed. 01/2024

Anti MPO (p-ANCA)

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16. BIBLIOGRAFIA

1. Basic QC Practices On-line Course; <http://www.Westgard.com>.
2. Boscato, LM. and Stuart, MC., 'Heterophilic antibodies: a problem for all immunoassays'. *Clin Chem*, 34, 1988, pp 27-33

Ed. 01/2024

DCM092-6

DCM092-6
Ed. 01/2024

Anti MPO (p-ANCA)

for routine analysis

Quantitative determination of IgG antibodies against myeloperoxidase (MPO) in human serum or plasma

IVD

LOT

See external label

2°C  8°C

 $\Sigma = 96$ tests

REF DKO092

1. INTENDED PURPOSE

For *In Vitro* Diagnostic Use

For Laboratory Professional Use

Anti MPO (p-ANCA) is a manual *in vitro* diagnostic device intended for the quantitative determination of IgG class autoantibodies against myeloperoxidase (MPO) in human serum or plasma from an adult population. Results are to be used in conjunction with other clinical and laboratory data as an aid in the diagnosis of certain autoimmune vasculitides such as Wegener's granulomatosis.

2. CLINICAL SIGNIFICANCE

Anti-neutrophilic-cytoplasm antibodies (ANCA) represent a group of autoantibodies directed towards the cytoplasmic components of the neutrophilic granulocytes and monocytes. The classical methods for the determination of ANCA are the immunofluorescent methods. With these indirect immunofluorescence techniques two main patterns are recognized, a cytoplasmic (c-ANCA) and a perinuclear (p-ANCA) type.

The main antigen for the c-ANCA is the proteinase 3 (PR3), which is a serine proteinase of the present in primary granules. Antibodies of p-ANCA positive sera are mainly directed to myeloperoxidase (MPO). Antibodies to other antigens e.g. lactoferrin, elastase, cathepsin-G and also lysozyme often result in a similar p-ANCA pattern. Beside different untypical variants of p-ANCA IF patterns granulocyte specific antinuclear antibodies (GS-ANA) is indistinguishable from p-ANCA. This makes a clear interpretation and classification of the IF patterns difficult.

Therefore, every positive IF-ANCA findings esp. p-ANCA should be differentiated by ELISA techniques using purified antigens. PR3- ANCA and MPO-ANCA are reliable serologic markers in the diagnostics of vasculitides.

PR3- ANCA is the classical autoantigen in Wegener's granulomatosis with a clinical specificity of more than 95%. c-ANCA is documented to be present in different diseases. Anti-MPO antibodies are highly specific for idiopathic and vasculitis associated crescentic glomerulonephritis and also for classic polyarteritis nodosa, Churg-Strauss syndrome and the polyangitis overlap syndrome without renal involvement. With respect to sensitivity, either MPO or PR-3 antibodies were found in 77 to 100% of patients with idiopathic and vasculitis associated crescentic

glomerulonephritis. In WG, anti-MPO antibodies were detected only occasionally and generally in patients negative for PR-3 antibodies.

The MPO and PR-3 specific ELISA methods can provide an important confirmatory result for two of the more important of the identified antigens. ELISA is also useful for interpreting "difficult" samples by IFA such as those which exhibit several antibodies simultaneously or those with high background fluorescence.

3. PRINCIPLE OF THE METHOD

The Anti MPO (p-ANCA) assay is an indirect enzyme immunometric assay (ELISA) where anti- MPO (antibody) in the sample binds to human neutrophil myeloperoxidase coated into the microplate (solid phase). In the first step the antibodies in calibrators, controls or prediluted patient samples bind to the inner surface of the wells. After a 60-minute incubation, the microplate is washed to remove non-reactive serum components. In the second step an anti-human-IgG horseradish peroxidase (HRP) conjugated solution binds to the IgG class antibodies bound to the immobilised antigens. After a further incubation step, any excess enzyme conjugate which is not specifically bound is washed away.

A chromogenic substrate solution containing TMB is then dispensed into the wells. After a further incubation step, the enzyme HRP in the bound fraction reacts with the Substrate (H_2O_2) and the TMB Substrate and develops a blue colour that changes into yellow when the Stop Solution (H_2SO_4) is added. The colour intensity is directly proportional to the concentration of IgG antibodies in the sample.

The concentration of anti- MPO IgG antibodies in the sample is calculated through a calibration curve.

4. REAGENTS, MATERIALS AND INSTRUMENTATION

4.1. Reagents and materials supplied in the kit

1. Calibrators (5 vials, 1.2 mL each)
Phosphate buffer 0.1M, Sodium Azide <0.1%, BSA 3%
CAL0 REF DCE002/09206-0
CAL1 REF DCE002/09207-0
CAL2 REF DCE002/09208-0
CAL3 REF DCE002/09209-0
CAL4 REF DCE002/09210-0
2. Control (2 vials, 1.2 mL each)
Concentration of Control is indicated on the Certificate of Analysis. Phosphate buffer 0.1M, Sodium Azide <0.1%, BSA 3%
Negative control REF DCE045/09201-0
Positive control REF DCE045/09202-0
3. Sample Diluent (1 vial, 100 mL)
Phosphate buffer 0.1M, NaN₃ <0.1%
REF DCE053-0
4. Conjugate (1 vial, 15 mL)
Anti- human IgG conjugated with horseradish peroxidase (HRP). ProClin >0.0015%, BSA 0.1%
REF DCE002/09202-0
5. Coated Microplate (1 microplate breakable)
Anti- human neutrophil myeloperoxidase antibodies adsorbed on microplate REF DCE002/09203-0
6. TMB Substrate (1 vial, 15 mL)
H₂O₂-TMB (0.26 g/L) (avoid any skin contact)
ProClin <0.0015% REF DCE004-0
7. Stop Solution (1 vial, 15 mL)
Sulphuric acid 0.15M (avoid any skin contact)
REF DCE005-0
8. 10X Conc. Wash Solution (1 vial, 50 mL)
Phosphate buffer 0.2M pH 7.4. ProClin >0.0015%
REF DCE054-0

4.2. Materials required but not provided

Distilled water

4.3. Auxiliary materials and instrumentation

Automatic dispenser
Precision Pipetting Devices
Microplate reader (450 nm, 620-630 nm)

5. WARNINGS

- This kit is intended for *in vitro* use by professional persons only. Not for internal or external use in Humans or Animals.
- Use appropriate personal protective equipment while working with the reagents provided.
- Follow Good Laboratory Practice (GLP) for handling blood products.
-  Material of animal origin used in the preparation of the kit has been obtained from animals certified as healthy and the bovine protein has been obtained from countries not infected by BSE, but these materials should be handled as potentially infectious.
- Some reagents (conjugate and wash solution) contain small amounts of ProClin™ 300 (>0.0015%, <0.06%) as preservative. Avoid contact with skin or mucosa.

- Classification according to Regulation (EC) No. 1272/2008 [CLP]

Skin sensitivity, Category 1

Contains: ProClin 300

Warning

Hazard statements:

H317 - May cause an allergic skin reaction.

Precautionary statements:

P261 - Avoid breathing dust / fume / gas / mist / vapours / spray.

P280 - Wear protective gloves/ protective clothing / eye protection / face protection / hearing protection.

P321 - Specific treatment (see supplemental first aid instruction on this label).

P333+P313 - If skin irritation or rash occurs: Get medical advice/attention.

P362+P364 - Take off contaminated clothing and wash it before reuse.

- Some reagents (calibrators, controls and sample diluent) contain small amounts of Sodium Azide (NaN₃) <0.1% which may be toxic if ingested or absorbed through the skin or eyes; moreover, it may react with lead or copper plumbing to form potentially explosive metal azides. If you use a sink to remove the reagents, wash through with large amounts of water to prevent azide build-up.
- The Stop Solution consists of a diluted sulphuric acid solution. Sulphuric acid is poisonous, corrosive and can be toxic if ingested. To prevent chemical burns, avoid contact with skin and eyes.
- Avoid the exposure of reagent TMB/H₂O₂ to direct sunlight, metals or oxidants. Do not freeze the solution.

6. PRECAUTIONS

- Please adhere strictly to the sequence of pipetting steps provided in this protocol. The performance data represented here were obtained using specific reagents listed in this Instruction For Use.
- All reagents should be stored refrigerated at 2 – 8°C in their original container. Any exceptions are clearly indicated.
- Allow all kit components and specimens to reach room temperature (22 – 28°C) and mix well prior to use.
- Do not interchange kit components from different lots. The expiry date printed on box and vials labels must be observed. Do not use any kit component beyond their expiry date.
- If you use automated equipment, the user has the responsibility to make sure that the kit has been appropriately validated for its intended use/purpose.
- The incomplete or inaccurate liquid removal from the wells could influence the assay precision and/or increase the background. To improve the performance of the kit on automatic systems is recommended to increase the number of washes.
- It is important that the time of reaction in each well is held constant for reproducible results. Pipetting of samples should not extend beyond ten minutes to avoid assay drift. If more than 10 minutes are needed, follow the same order of dispensation. If more than one plate

is used, it is recommended to repeat the dose response curve in each plate

- Addition of the TMB Substrate solution initiates a kinetic reaction, which is terminated by the addition of the Stop Solution. Therefore, the TMB Substrate and the Stop Solution should be added in the same sequence to eliminate any time deviation during the reaction.
- Observe the guidelines for performing quality control in medical laboratories by assaying controls and/or pooled sera.
- Maximum precision is required for reconstitution and dispensation of the reagents.
- Samples microbiologically contaminated, highly lipemic, icteric or haemolysed should not be used in the assay.
- Plate readers measure vertically. Do not touch the bottom of the wells.
- **WARNING: the conjugate reagent is designed to ensure maximum dose sensitivity and may be contaminated by external agents if not used properly;** therefore, it is recommended to use disposable consumables (tips, bottles, trays, etc.). For divided doses, take the exact amount of conjugate needed and do not re-introduce any waste product into the original bottle. In addition, **for doses dispensed with the aid of automatic and semi-automatic devices,** before using the conjugate, it is advisable to clean the fluid handling system, ensuring that the procedures of washing, deproteinisation and decontamination are effective in avoiding contamination of the conjugate; **this procedure is highly recommended when the kit is processed using analysers which are not equipped with disposable tips.** For this purpose, Diametra supplies a separate decontamination reagent for cleaning needles.
- Fresh disposable tips must be used when pipetting assay reagents including samples, calibrators and controls to mitigate the risk of carryover contamination. Failure to do so may lead to invalid results.

7. REAGENT STORAGE AND STABILITY

Store the kit at 2 – 8°C in the dark.

- Once opened, the calibrators are stable at 2 – 8°C for 6 months.
- The diluted wash solution is stable for 30 days at 2 – 8°C.

Important note: open the bag containing the Coated Microplate only when it is at room temperature and close it immediately after use.

8. SAMPLE COLLECTION AND STORAGE

The assay should be performed using serum or plasma samples.

Sample Storage	Duration
2 – 8 °C	5 days
-20°C	6 months

9. PROCEDURE

9.1. Preparation of Calibrators and Controls

Before use, mix for 5 minutes with a rotating mixer.

The calibrators are ready for use and have the following concentration:

	C ₀	C ₁	C ₂	C ₃	C ₄
AU/mL	0	10	20	40	160

The controls are ready to use; the concentration is printed on the label.

9.2. Preparation of the Conjugate

The conjugate is ready to use.

9.3. Preparation of the Wash Solution

Dilute the content of the vial “10X Conc. Wash Solution” with distilled water to a final volume of 500 mL prior to use. For smaller volumes respect the 1:10 dilution ratio.

It is possible to observe the presence of crystals within the concentrated wash solution; in this case mix at room temperature until the complete dissolution of crystals. For greater accuracy, dilute the whole bottle of concentrated wash solution to 500 mL, taking care also to transfer crystals completely by rinsing of the bottle, then mix until crystals are completely dissolved.

9.4. Preparation of Samples

The determination of Anti- MPO (p-ANCA) can be performed in human serum or plasma samples.

All serum and plasma samples must be pre-diluted with sample diluent 1:100; for example, 10 µL of sample may be diluted with 990 µL of sample diluent.

Fasting samples are not necessary and no special sample preparations are required.

Collect blood by venepuncture into vacutainers and separate serum (after clot formation) or plasma from the cells by centrifugation. Testing of heat-inactivated sera is not recommended.

Store the sample at -20°C if the determination is not performed on the same day of the sample collection. Before using, mix gently, for 5 minutes, with a roller mixer.

9.5. Procedure

- **Allow all reagents to reach room temperature (22 – 28 °C) for at least 30 minutes.** At the end of the assay, immediately store the reagents at 2 – 8°C: avoiding long exposure to room temperature.
- Unused coated microwell strips should be released securely in the foil pouch containing desiccant and stored at 2 – 8°C.
- To avoid potential microbial and/or chemical contamination, unused reagents should never be transferred into the original vials.
- As it is necessary to perform the determination in duplicate in order to improve accuracy of the test results, prepare two wells for each point of the calibration curve (C₀-C₄), two for each Control, two for each sample, one for Blank.

Reagent	Calibrator	Sample/ Control	Blank
Calibrator C ₀ -C ₄	100 µL		
Control		100 µL	
Diluted sample		100 µL	

Incubate 60 minutes at room temperature (22 – 28°C).
Remove the contents from each well, wash the wells 3 times with 300 µL of diluted wash solution.

Important note: during each washing step, gently shake the plate for 5 seconds and remove excess solution by tapping the inverted plate on an absorbent paper towel.

Automatic washer: if you use automated equipment, wash the wells at least 5 times.

Conjugate	100 µL	100 µL	
-----------	--------	--------	--

Incubate 30 minutes at room temperature (22-28°C).
Remove the contents from each well, wash the wells 3 times with 300 µL of diluted wash solution.

Washing: follow the same indications of the previous point.

TMB Substrate	100 µL	100 µL	100 µL
---------------	--------	--------	--------

Incubate at 22 – 28°C for 15 minutes in the dark.

Stop Solution	100 µL	100 µL	100 µL
---------------	--------	--------	--------

Gently shake the microplate.
Read the absorbance (E) at 450 nm against a reference wavelength of 620-630 nm or against Blank within 5 minutes.

10. QUALITY CONTROL

Good Laboratory Practice (GLP) requires the use of quality control specimens in each series of assays in order to check the performance of the assay. Controls should be treated as unknown samples, and the results analysed with appropriate statistical methods.

The kit controls provided in the kit should be tested as unknowns and are intended to assist in assessing the validity of results obtained with each assay plate.

The mean concentration of each control level is documented in the QC report included with each kit. These mean concentration levels are determined over several assays which are run in duplicate in multiple locations across each plate.

DiaMetra recommends the users to maintain graphic records of the control values generated with each assay run, including the running means, SDs and %CVs. This information will facilitate the controls trending analysis relating to the performance of current and historical control lots relative to the supplied Quality Control data. The trending will assist in the identification of assays which give control values significantly different from their average range.

When interpreting control data, users should note that this product was designed and developed as a manual product. The range stated on the QC certificate should be appropriate for assays that are performed manually and with strict adherence to the Assay Procedure described above. It is recognised by Quality Control professionals, that as a result of differences in conditions and practices, there will always be variability in the mean values and precision of control measurements between different laboratories¹.

11. CALCULATION OF RESULTS

A variety of data reduction software packages are available, which may be employed to generate the mean calibration curve and to calculate the mean concentrations of unknown samples and controls. A 4-parameter logistic (4PL) curve fit, **including Calibrator 0 is required**. Smoothed-Spline Approximation and log-log coordinates are also suitable.

Alternatively, a calibration curve may be prepared on semi-log graph paper by plotting mean absorbance on the Y-axis against concentration of analyte on the X-axis. Calibrator 0 should be included in the calibration curve. Read the mean absorbance value of each unknown sample off the curve.

In order for the test results to be considered valid, all of the criteria listed below must be met. If any of these are not met, the test should be considered invalid and the assay repeated:

- The absorbance of the prediluted MPO IgG Positive must be greater than the absorbance of the prediluted Negative Control.
- The Negative and Positive Control are intended to monitor for substantial reagent failure and they will not ensure precision at the assay cutoff.
- This test is only valid if the optical density at 450 nm for Positive Control (1) and Negative Control (2) as well as for the Calibrator S0-S5 complies with the respective range indicated on the Quality Control Certificate enclosed to each test kit: If any of these criteria is not met, the results are invalid and the test should be repeated.

12. EXPECTED VALUES

In a normal range study with serum samples from healthy blood donors the following ranges have been established with the Anti MPO (p-ANCA) tests:

	Anti MPO (p-ANCA)
Cut-off	20 AU/mL

Please pay attention to the fact that the determination of a range of expected values for a "normal" population in a given method is dependent on many factors, such as specificity and sensitivity of the method used and type of population under investigation. Therefore, each laboratory should

consider the range given by the Manufacturer as a general indication and produce their own range of expected values based on the indigenous population where the laboratory works.

Positive results should be verified concerning the entire clinical status of the patient. Also every decision for therapy should be taken individually. It is recommended that each laboratory establishes its own normal and pathological ranges of serum Anti-MPO.

13. PERFORMANCE CHARACTERISTICS

Representative performance data are shown. Results obtained at individual laboratories may vary.

13.1. Specificity

Comparison test against a commercial reference kit, performed on 32 sera (8 of them positive sera and 24 negative sera) showed a 100% specificity.

13.2. Sensitivity

Comparison test against a commercial reference kit, performed on 32 sera (8 of them positive sera and 24 negative sera) showed a 100% sensitivity.

13.3. Detection Limit

The lowest concentration of anti MPO antibodies that can be distinguished from the Calibrator 0 is about 0.73 AU/mL with a confidence limit of 98%.

13.4. Precision and reproducibility

13.4.1. Intra-Assay

Within run variation was determined by replicate the measurements of three different sera with values in the range of the calibration curve. The within assay variability is $\leq 6.1\%$.

13.4.2. Inter-Assay

Between run variation was determined by replicate the measurements of two different control sera with different lots of kits and/or different mix of lots of reagents. The between assay variability is $\leq 12.3\%$.

14. LIMITATIONS OF USE

- The presence of immune complexes or other immunoglobulin aggregates in the patient sample may cause an increased level of non-specific binding and produce false positives in this assay.
- As in the case of any diagnostic procedure, results must be interpreted in conjunction with the patient's clinical presentation and other information available to the physician.
- The performance characteristics of this assay have not been established in a paediatric population.
- Heterophilic antibodies in human serum can react with reagent immunoglobulins, interfering with *in vitro* immunoassays². Patients routinely exposed to animals or to animal serum products can be prone to this interference and anomalous values may be observed.

15. WASTE MANAGEMENT

Reagents must be disposed of in accordance with local regulations.

All materials that have come into contact with samples and reagents must be disposed of in accordance with country, state and local regulations.

16. BIBLIOGRAPHY

1. Basic QC Practices On-line Course; <http://www.Westgard.com>.
2. Boscato, LM. and Stuart, MC., 'Heterophilic antibodies: a problem for all immunoassays'. *Clin Chem*, 34, 1988, pp 27–33

17. REVISION IDENTIFIER

Additions or changes to the IFU are indicated by grey highlighting.

18. PRODUCT COMPLAINTS AND TECHNICAL SUPPORT

For a patient/user/third party in the European Union and in countries with similar regulatory regime (Regulation 2017/746/EU on IVD Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national regulatory authority.

The manufacturer can be contacted through their customer service or technical support team. The contact details can be found below and on the company website: www.diametra.com.

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